


## MICROBIOLOGY AND MISCELLANEOUS SPECIMEN HANDLING INSTRUCTIONS

Test Required	Specimen	Notes	Storage Requirements
Chlamydia/GC Nucleic Acid Testing	Swab cervix or urethra	Use special Aptima® unisex swab for Chlamydia and GC testing. A single swab is used to test for Chlamydia and GC. Clearly indicate the specimen site.	Room temperature
Chlamydia/GC Nucleic Acid Testing	Urine	Requires specific patient preparation / instructions to collect a first-void urine specimen. A single specimen is used to test for Chlamydia and GC.	Refrigerate immediately
Culture – Routine Bacterial	Swab	Please specify 'source' and diagnosis for all swabs. If fungal culture also requested please submit a separate specimen.	Room temperature
Herpes – Culture	Swab	Submit BCCDC Herpes swab with a completed BCCDC requisition.	Refrigerate immediately
Joint/Synovial Fluid Analysis	Fluid	Collect fluid and process following the instructions in the LifeLabs Synovial/Joint Fluid Collection Kit.	Room temperature
Pertussis – Culture	Nasopharyngeal swab	Special collection kit required from BCCDC. Submit With a completed BCCDC requisition.	Refrigerate immediately
Semen Analysis	Semen	Specimen requires specific patient preparation/instructions. Please send patient to laboratory.	Room temperature
Sputum - Culture and Sensitivity (C&S)	Sputum	First morning specimen preferred.	Refrigerate
Stool – <i>Clostridium difficile</i> (Cdiff)	Stool	Specimen requires specific patient preparation / instructions. Please send patient to laboratory.	Refrigerate immediately
Stool – Culture and Sensitivity, WBC	Stool in Cary Blair liquid	Specimen requires specific patient preparation / instructions. Please send patient to laboratory.	Refrigerate
Ova and Parasite (O&P)	Stool in SAF preservative	Specimen requires specific patient preparation / instructions. Please send patient to laboratory.	Room temperature
Stool - Occult Blood	Stool	Specimen requires specific patient preparation/ instructions as outlined in collection kits. Please send patient to laboratory.	Room temperature
Urine - Boric Acid/ Urine preservative	Urine	Requires specific patient preparation / instructions for collection of a midstream urine specimen.	Room temp or Refrigerate
Urine – Culture (not in preservative)	Urine	Requires specific patient preparation / instructions for collection of a midstream urine specimen.	Refrigerate immediately
Urine - Pregnancy Test	Urine	First morning specimen or random specimen.	Refrigerate.
Urine – Routine Urinalysis	Urine	Fresh random urine collection. A minimum amount of 12 mLs is required.	Refrigerate immediately
Vagina - Initial presentation	Swab  2 <sup>nd</sup> swab.	Stained smear for Yeast & Bacterial Vaginosis (BV).  Trichomonas test performed when requested.	Room temperature
Vagina – Recurrent/Chronic	3 Swabs (one each for smear, culture & Trichomonas)	Stained smear for Yeast and BV. Yeast culture, Trichomonas test and Genital culture.	Room temperature
Vaginal-anal (Group B Strep screen only)	Swab	Group B Streptococcus screen only	Room temperature

- Label all specimens clearly with the patient first and last name, source (if applicable) and the date collected.
- All specimens should be forwarded for processing on the collection date.
- If there are any delays, contact the laboratory.

	<b>MICROBIOLOGY AND MISCELLANEOUS SPECIMEN HANDLING INSTRUCTIONS</b>		Page 1 of 1
	<b>MASTER INDEX #</b>	<b>LOP-PR-003</b>	<b>VERSION DATE</b> JAN-16-2012

Printed copies are uncontrolled. Printed versions should be checked and compared to current version found on MyLab BC